# INSTRUCTIONS FOR USE

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- EN** Medical compression garments
- DE** Medizinische Kompressionsversorgung
- FR** Vêtement de compression médical
- ES** Prenda de compresión
- PT** Peça de vestuário de compressão médica
- IT** Indumenti medicali compressivi
- NL** Medische compressiekous
- SV** Medicinskt kompressionsplagg
- NO** Medisinsk trykkbandasje
- DA** Medicinsk støttestrømpe
- FI** Lääkinnällinen kompressiovaate
- IS** Þrýstifatnaður í lækningaskyni
- ET** Meditsiinilised kompressioonrõivad
- LT** Medicininė kompresinė apranga
- LV** Medicīniskās kompresijas apģērbs
- PL** Medyczna odzież uciskowa
- HU** Gyógyászati kompressziós ruhadarab
- RO** Produs de compresie Medical
- CZ** Zdravotní Kompresní Výrobky
- SL** Medicinsko kompresijsko oblačilo
- RU** Медицинская компрессионная одежда
- AR** رداء ضاغط طبي
- TR** Tibbi Kompresyon Giysisi
- HE** בגד לחץ רפואי

<b>EN</b>	<b>FIBER CONTENT</b>
<b>DE</b>	<b>MATERIALZUSAMMENSETZUNG</b>
<b>FR</b>	<b>COMPOSITION</b>
<b>ES</b>	<b>COMPOSICIÓN</b>
<b>PT</b>	<b>TEOR DE FIBRA</b>
<b>IT</b>	<b>COMPOSIZIONE DEL MATERIALE</b>
<b>NL</b>	<b>SAMENSTELLING VAN DE STOF</b>
<b>SV</b>	<b>FIBERINNEHÅLL</b>
<b>NO</b>	<b>INNOLD</b>
<b>DA</b>	<b>MATERIALESAMMENSÆTNING</b>
<b>FI</b>	<b>KUITUSISÄLTÖ</b>
<b>IS</b>	<b>TREFAJAINNIHALD</b>

<b>ET</b>	<b>KOOSTIS</b>
<b>LT</b>	<b>PLUOŠTO SUDĖTIS</b>
<b>LV</b>	<b>AUDUMA SASTĀVS</b>
<b>PL</b>	<b>SKŁAD</b>
<b>HU</b>	<b>ANYAGÖSSZETÉTEL</b>
<b>RO</b>	<b>CONTIN FIBRE</b>
<b>CZ</b>	<b>SLOŽENÍ MATERIÁLU</b>
<b>SL</b>	<b>VSEBNOST VLAKEN</b>
<b>RU</b>	<b>СОСТАВ ВОЛОКОН</b>
<b>AR</b>	<b>محتوى النسيج</b>
<b>TR</b>	<b>FIBER İÇERİK</b>
<b>HE</b>	<b>תכולת הסיבים</b>

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## Dear customer,

Thank you for choosing JOBST®, the medical compression garment that incorporates gradient compression for the management of venous and lymphatic diseases. JOBST® offers you a wide selection of products for these indications.

### 1. Directions for use

#### Tips for wearing JOBST® Medical Compression garment:

To get the best medical result from your JOBST® garment, wear it regularly and consistently.

Ideally you should put on your compression garment in the morning, immediately after getting up.

You may experience some difficulty putting on the garment in the beginning, but this will become easier after a few days of use.

When applying the compression garments, please do not pull on the edge of the garments instead follow these recommendations:

Turn the arm sleeve inside out and fold the front part of the compression arm sleeve inward.

Use the other hand to pull the inside-out arm sleeve as far over the other arm as possible.

Then fold the upper end outward and pull the compression arm sleeve gradually over the elbow and up to the axilla.

TIP: Hold on to some support (e.g. a banister, door handle) with the arm to be dressed.

The compression arm sleeve fits correctly when it covers the arm evenly without forming creases.

The two components of arm sleeves with a separate glove must overlap. Arm sleeves may be secured by:

- Shoulder strap
- Bra strap
- Adhesive silicone band

For applying the compression glove please adhere to the following recommendations: Slip fingers into the glove starting at the wrist.

Position the glove so that it is comfortable around the palm and thumb. To assure a precise fit, work the glove into the web spaces of your hand.

#### Special notes:

- Caution: long fingernails and jewelry may damage the garments when putting them on. For easier application use rubber gloves or a donning aid.
- Do not bring the compression garments into contact with greasy ointments, body lotions or similar substances unless recommended by JOBST®.
- Loose threads or loops should not be cut off, since this may cause holes or runs to form in the garment.
- Intended to be used by a single patient only.

### 2. Care instructions

JOBST® compression garments should, if at all possible, be washed daily, or at least every other day.

For this purpose, you may use a regular mild detergent. Do not use laundry additives such as fabric softener, optical brightener or stain remover. Rinse your compression garment well. Roll up the wet compression garments in a towel after washing and press out excess moisture. Do not wring.

Do not dry the compression garment on a radiator or in direct sunlight.

For tumble dryer compatibility please check your compression stocking's product label.

#### Silicone band care:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign material.

### 3. Intended use and indications

Every JOBST® compression product has a specific compression level designed to help specific venous and lymphatic conditions.

The compression level is printed on the package label.

The required compression class must be determined by your physician.

Medical compression garments are used for the management of the following indications:

- Primary and secondary arm lymphedema in ISL stages I, II, and III
- Lipedema

- Paget-von-Schrötter syndrome (axillary phlebothrombosis)
- Klippel-Trenaunay symptom complex
- Rheumatoid Arthritis
- Hypertrophic scars

## 4. Contraindications

JOBST® compression garments should not to be worn in cases of:

- Advanced arterial insufficiency including ischemia
- Uncontrolled congestive heart failure
- Untreated septic phlebitis
- Phlegmasia coerulea dolens
- Incompatibility to fabric
- Open wounds not covered with an appropriate dressing

## 5. Cautions

- Weeping dermatosis
- Cutaneous infections
- Open wounds must be covered with an appropriate dressing before the compression garment is put on.
- Severely compromised skin sensibility and impaired sensitivity of the limb
- Advanced peripheral neuropathy
- Immobility (confinement to bed). Not recommended to wear during bed rest.
- Conditions in which increased venous and lymphatic return is not desired.
- Appropriate hygiene and skin care should be applied.
- Ill-fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage.
- The attending physician decides whether compression in general can be applied to patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome (CRPS, M. Sudeck), malignant lymphedema, gangrene.
- Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- Patient measurements should be taken by a trained person to determine the correct product fit.
- When re-ordering a garment after 6 months we recommend remeasuring the patient as the limb circumference might have changed.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using

the product and contact your healthcare professional.

- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent state authority.
- Caution applicable for made-to-measure products: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

## 6. Directions for storage / Guarantee

A maximum storage period of up to 36 months applies for serial products.

This is followed by a wearing period of not more than 6 months for the compression stocking.

The expiry date for storage is printed on the package label with the symbol of an hourglass.

Made-to-measure products are products intended for immediate use and no expiry date for storage is indicated on these.

Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of six months at most both for serially produced sizes and made-to-measure products. This assumes correct handling (e.g. product care, donning and doffing).

Please protect your compression garments from direct exposure to sunlight, heat and moisture. We recommend storage at room temperature.

If the disease persists after 6 months, please consult your attending physician concerning the possible need of a new garment.

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BSN-JOBST GmbH
Essity UK Ltd, Southfields Road, DUNSTABLE, Beds, LU6 3EJ
Australian Distributor: BSN medical (Aust.) Pty Ltd